CalmDown

by KRUUSE®

GB

CalmDown by KRUUSE® Post-Op Button, cats

Help cats feel safe, comfortable and relaxed.

Having to wear a collar can be stressful for cats. It interferes with their vision, smell and awareness.

The CalmDown by KRUUSE® Post-Op Button comforts cats and has a soothing effect because it mimics the natural comforting pheromones that a mother cat produces to soothe her kittens.

The button contains a scientifically tested pheromone that helps soothe cats in stressful situations.

It is non-invasive, easy to use and works for at least 10 days

The button is suitable for cats of all ages.
It can be used with all plastic collars.

Our click technology means you can simply click the button on the collar and let tranquillity settle in.

INSTRUCTIONS FOR USE:

1. Choose the appropriate collar for your pet.
2. Find the pre-punched holes in the centre of the collar.
If there are no pre-punched holes in the collar, use the CalmDown by KRUUSE® Hole Applicator to create holes in the collar. Please follow the instructions on the packaging for the Hole Applicator.
3. Ensure the button is placed on the inside of the collar, with the logo facing inward towards the cat.
4. Calming pheromones are released as soon as the button is inserted.

Calming pheromone: 2%

Inert ingredients: 98%

Weight: 2.0 g.

SAFETY INSTRUCTIONS:

For animal use only. Wash hands thoroughly after contact with the button. If skin irritation occurs, seek medical advice and bring along the packaging. Always monitor the cat and seek veterinary attention if the cat ingests the product. Ensure that the product is stored safely, out of reach of children and away from food, animal feed, medicine or similar items. Store in a dry place. Dispose of the product and packaging in accordance with local regulations.

DK

CalmDown by KRUUSE® Post-Op Knap, katte.

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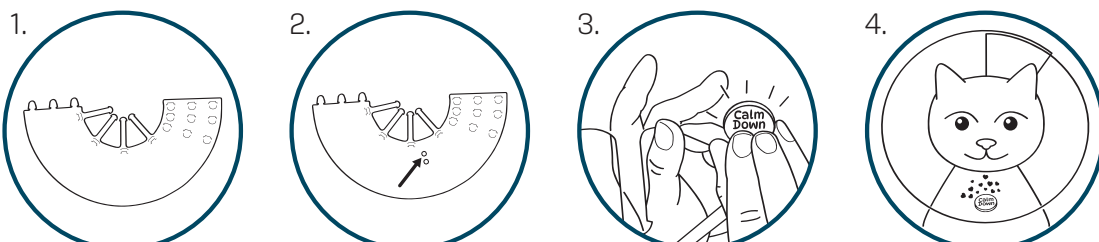
by KRUUSE®

NO

CalmDown by KRUUSE® Postoperativ knapp for katter.

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